

12hr

Comforting, penetrating, soothing warmth for hours

DEEP HEAT[®]

Period Pain Patch

Deep Heat Period Pain Patch is a long lasting deep penetrating heat therapy which provides soothing relief from muscular pain and menstrual cramps. This comfortable heating patch can be worn unnoticed under clothing as you go about your daily routine.


DIRECTIONS

- Open pouch to activate warming patch.
- Remove protective film.
- Apply patch directly to the area of discomfort, adhesive side towards clothing for delicate or aging skin.
- For best contact apply to dry, flat area, free of moisture, oils or lotions.
- To remove – slowly peel off the patch.
- Patch will not re-stick well once removed.

WARNINGS

- To reduce the risk of burns or blisters, attach the adhesive side of the patch to your clothing over the area of discomfort.
- As with any heat product, this patch has the potential to cause skin irritation or burns, particularly to the sensitive skin of the elderly.
- Do not use for more than 12 hours in any 24 hour period on the same area.
- Discontinue use if skin sensitivity or irritation occurs, or if patch gets uncomfortably hot.
- Keep out of reach of children and pets.
- For external use only. If swallowed, get medical help or contact a Poison Control Center. If contents come in contact with skin or eyes, rinse immediately with water.

Patch shaped & heating cells designed for ultimate comfort

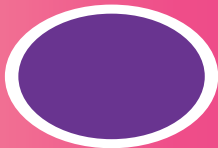

IMPORTANT: READ INFORMATION BELOW BEFORE USE. APPLY PATCH TO CLOTHING DO NOT USE

- if you are diabetic
- with sustained pressure against patch
- if allergic to adhesives

APPLY PATCH TO CLOTHING. PRODUCT IS AIR-ACTIVATED. DO NOT OPEN UNTIL READY TO USE.

DO NOT USE

- while sleeping
- if you are diabetic
- on face or mucous membranes
- on wounds or broken skin
- on areas of bruising or swelling
- on area of the body where heat cannot be felt
- on infants or children
- with topical medications
- with electric heating devices or other heat sources
- with sustained pressure against patch
- if allergic to adhesives

CAUTION

- Consult a doctor before use if you have poor circulation, nerve damage, a skin condition or are pregnant.
- Supervision required when being used by persons unable to apply/remove the product on their own; especially when being used by elderly, handicapped or disabled persons.

WHEN USING THIS PRODUCT

- do not use if product is punctured or torn
- do not cut, massage, or fold the patch
- do not bandage
- do not microwave or attempt to reheat product

STORAGE AND HANDLING

- Avoid exposure to direct sunlight or heat
- Store in a dry place

HEAT-GENERATING MATERIALS

Iron Powder, Activated Carbon, Vermiculite, Industrial Salt (Potassium Chloride), Water.



Mentholatum SA (Pty) Ltd. 1st Floor,
Silverberg Terrace, Steenberg Office Park,
Silverwood Close, Tokai, 7945.
Telephone: (021) 702 0620
www.deepheatsa.co.za
Made in China

